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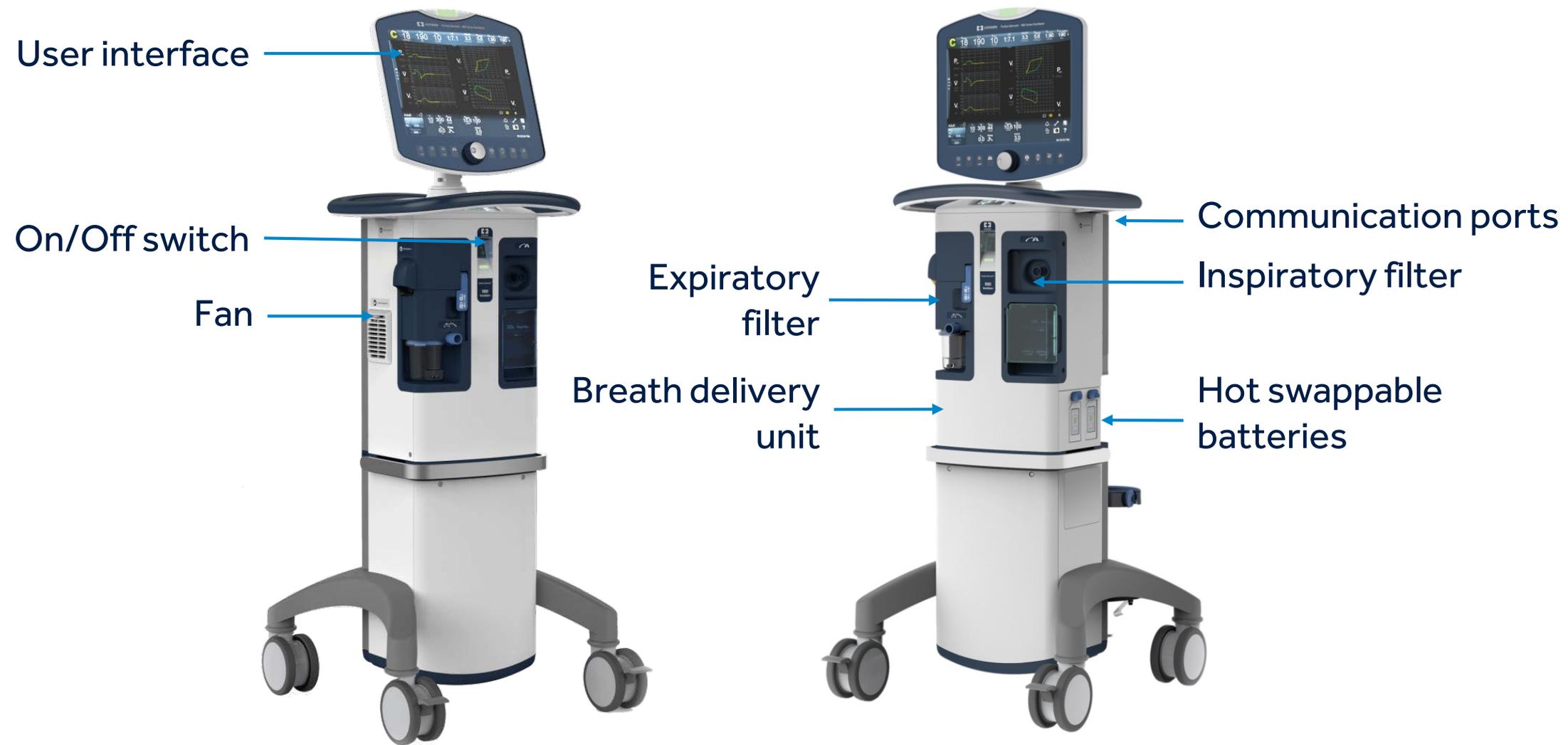
PB 980


GUI

POSITIONING THE USER INTERFACE FOR EASE OF VIEWING

THE USER INTERFACE


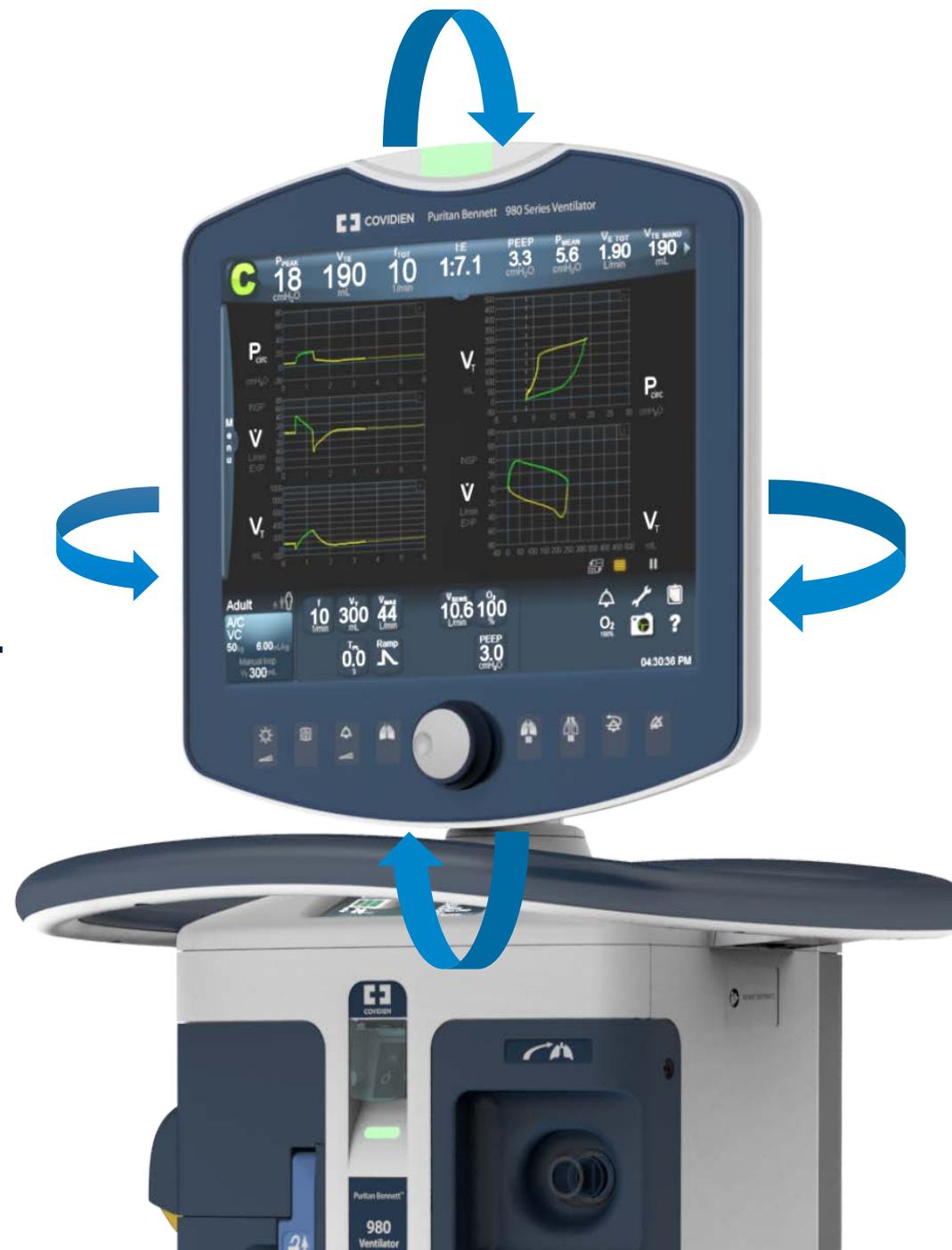
The GUI incorporates a 15" display that rotates throughout a 170° angle about a vertical axis in either direction. The GUI can also be tilted up to 45° from vertical.

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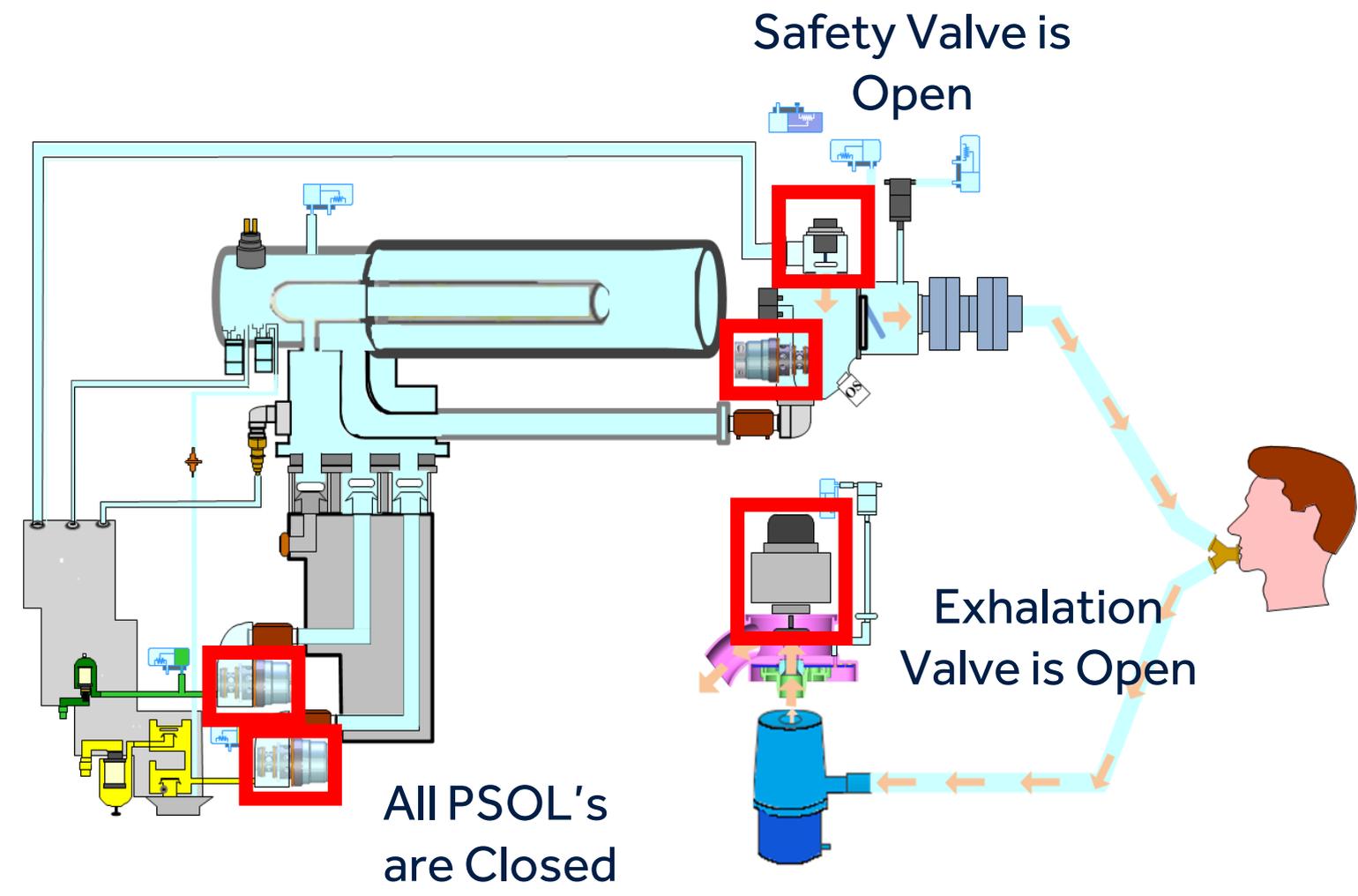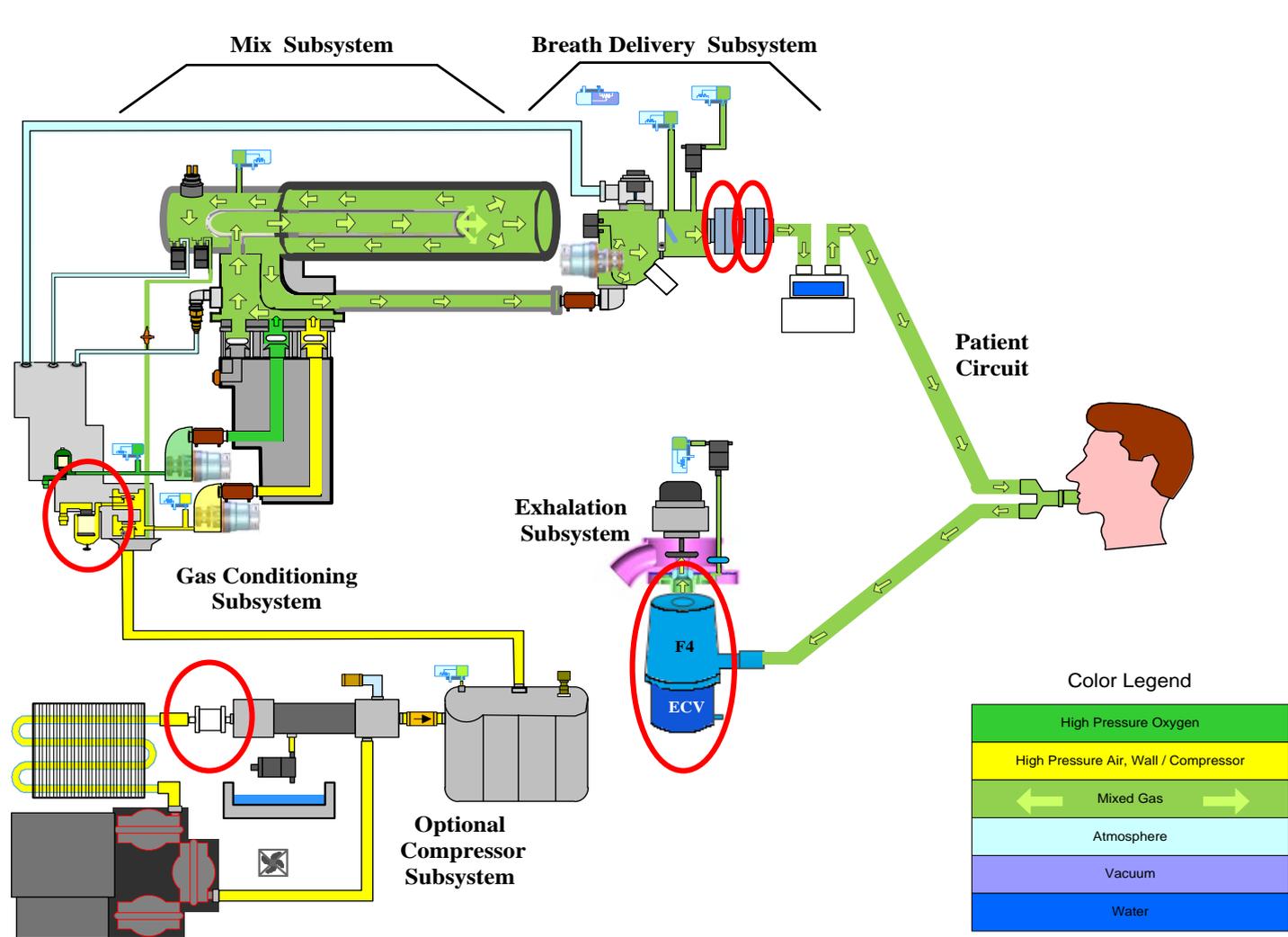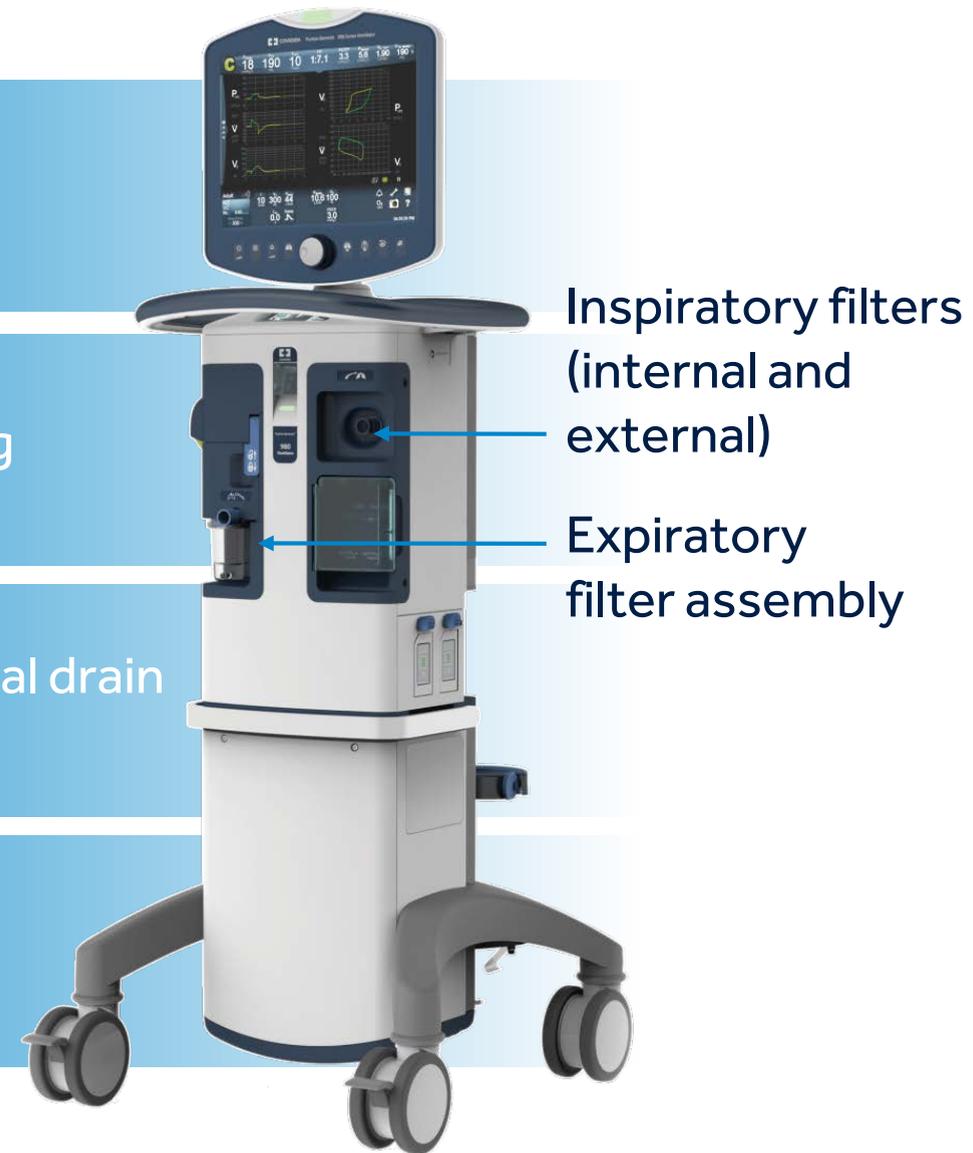
Three filters help reduce the spread of pathogens.

Expiratory filter is heated to keep the gas that flows through it from cooling to the dew point and creating condensate in the filter.

Reusable and disposable filter options are available.* An optional drain bag is available for managing condensate.

Pediatric-adult and neonatal applications use different expiratory filter configurations.

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4-074601-00
4-074600-00

FILTER INHALATION SPU X12
RE/FLEX FILTER 22MM X1

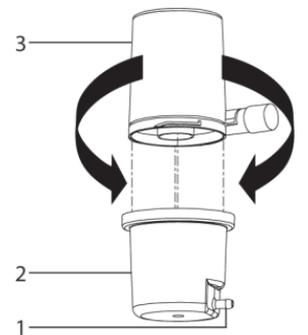
8 500,00 RUR
8 500,00 RUR

10043551
10063033
10063031

980 PAED/ ADT EXP FILTER DISP X12
980 PAED/ ADT EXP FILTER REUSE X1
980 CONDENSATE VIAL X1

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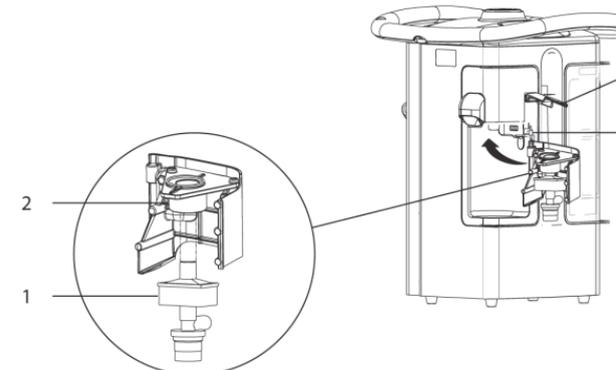
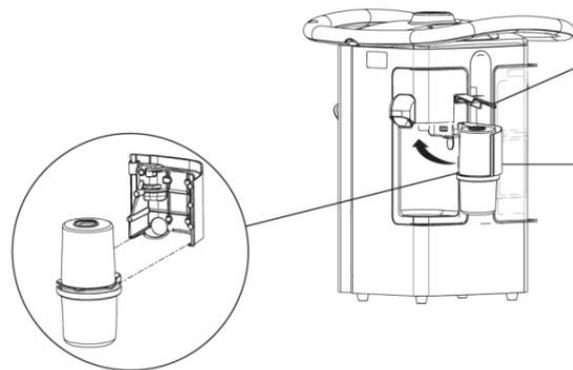
10086048
10097468

980 EVQ REPROCESSING SENSOR X6
980 EXH VALVE FLOW SENSOR X1

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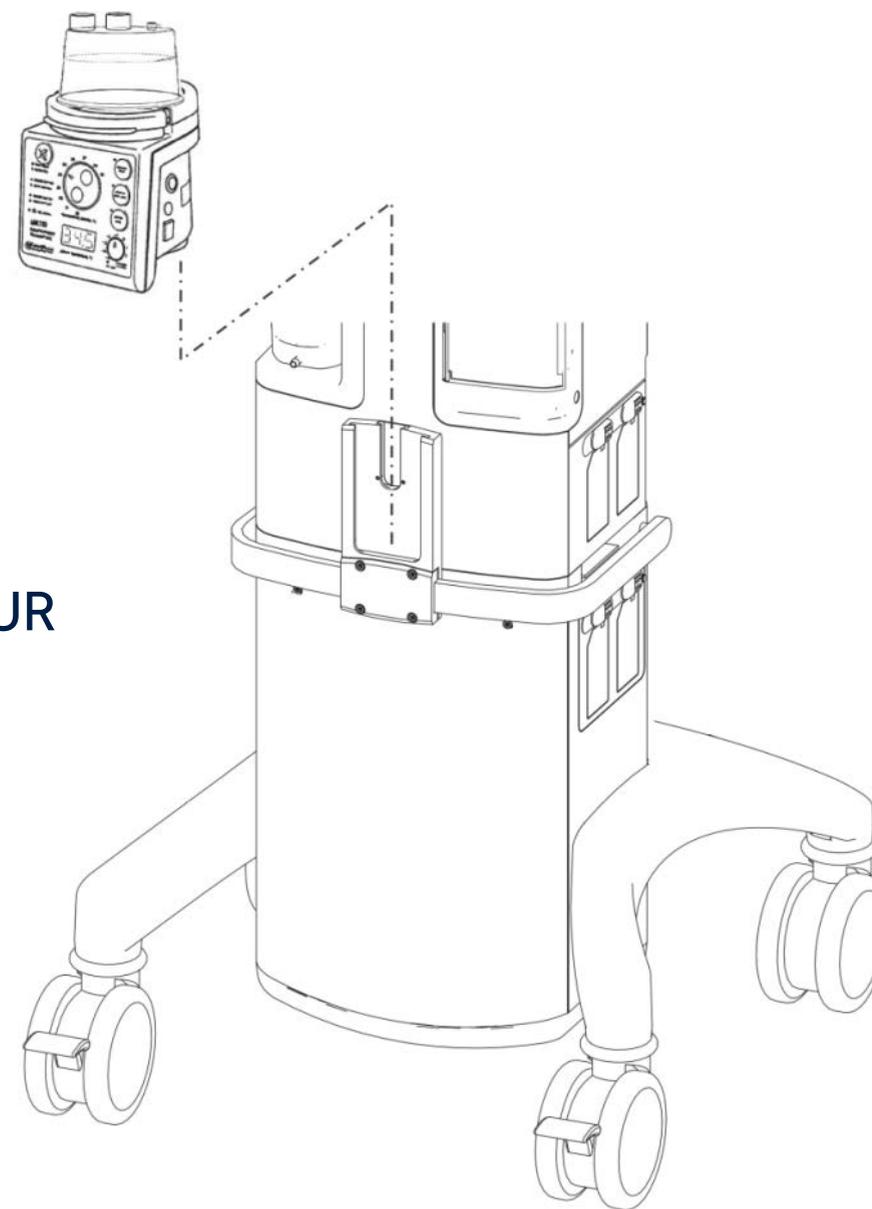
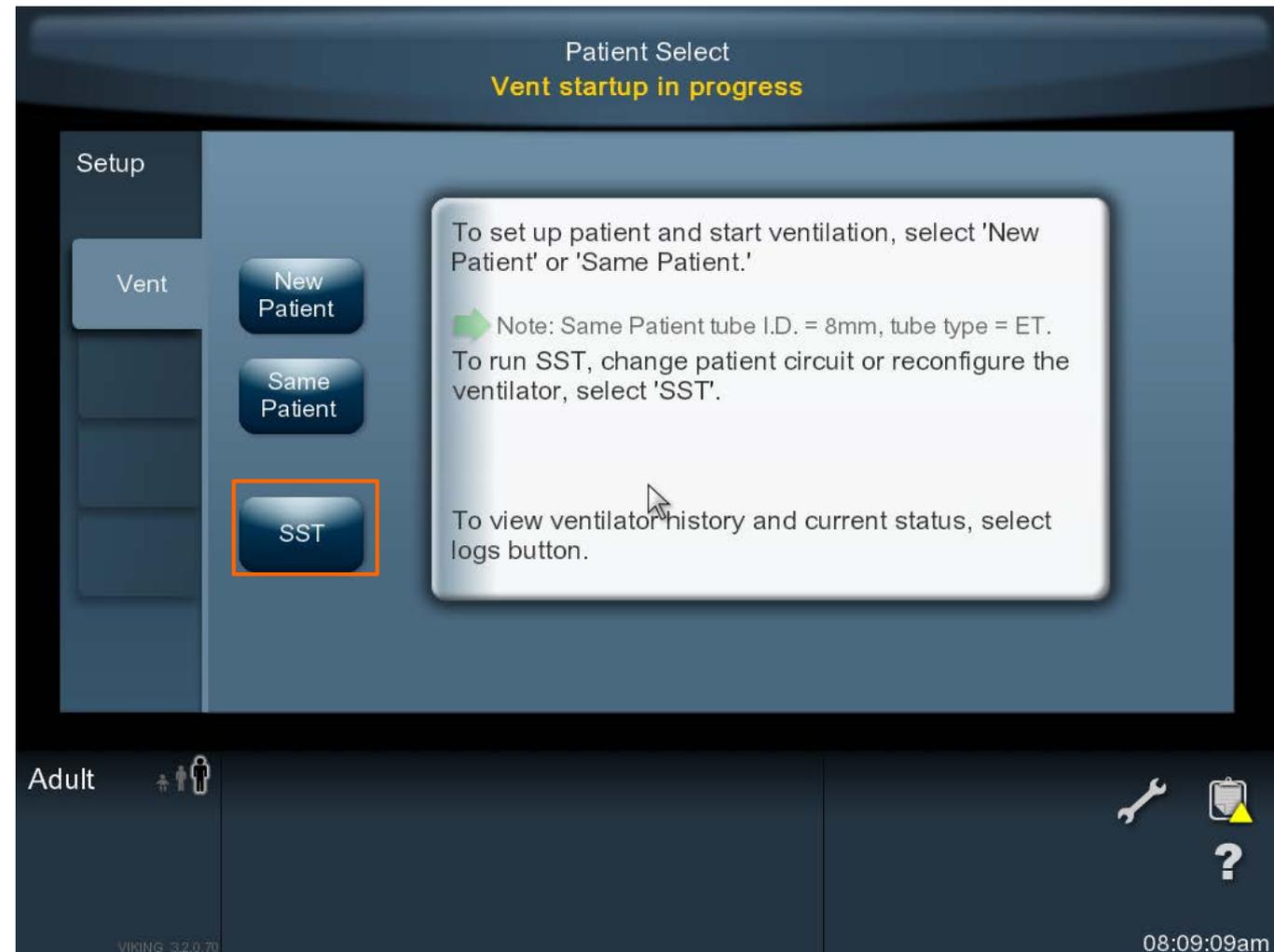
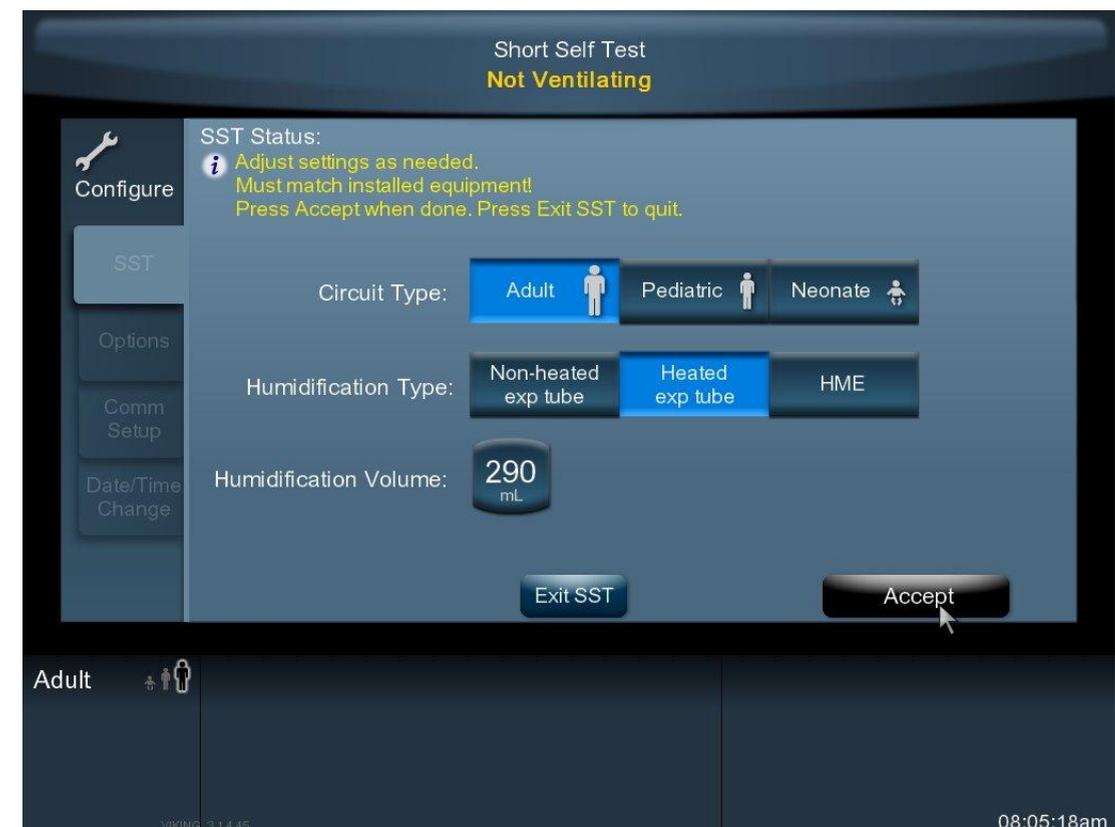
✓ Impacts accuracy of breath delivery and spirometry

- The SST is a way of calibrating the ventilator to the circuit and humidifier you are using. It impacts both breath delivery and spirometry.
- The power-on self test (POST) does basic testing of the system. The SST expands that testing.
It is the best way to prepare the ventilator for patient use.

✓ Run SST following a circuit change, change in circuit configuration (including circuit type, additions and removal of water traps and accessories, humidifiers, proximal flow sensor) and/or every 15 days.


✓ In the event that the patient is connected without ventilation parameters being specified, the ventilator enters Safety PCV, a safe mode of ventilation. Complete the parameter selection to exit Safety PCV.

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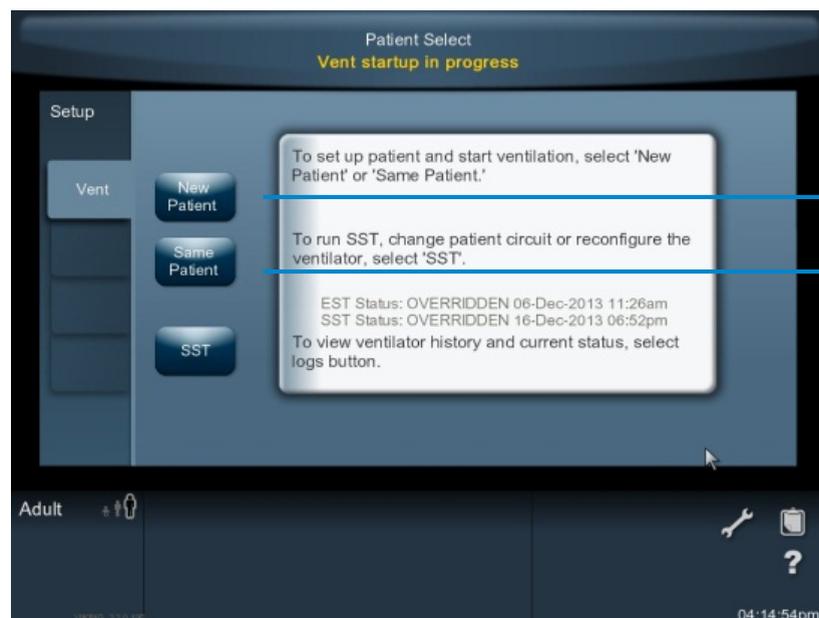- ✓ You can choose to set up a new patient or the same patient.
- ✓ In New Patient setup, you can choose a full manual setup or the Quick Start feature.

New patient	Manual setup	Quick start
<p style="text-align: center;">↑</p> <p style="text-align: center;">↓</p>	<ol style="list-style-type: none"> 1. Select PBW or Gender and Height 2. Select ventilation type—Invasive or Noninvasive (NIV) 3. Select a Mode 4. Select Mandatory, Spontaneous and Trigger types 5. Set the Primary settings 6. Touch Accept or Accept ALL to confirm the change(s) 	<ol style="list-style-type: none"> 1. Touch New Patient. 2. Enter PBW or gender and height. 3. Touch Quick Start. 4. Connect the circuit wye adapter to the patient's airway or interface connection.
<p style="text-align: center;">Same patient</p>	<p>The ventilator will be ready to start ventilating at the settings in place at power down.</p>	

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There are several ways to make a settings change after initial setup:

Swipe the **Menu** tab on the left margin of the touch screen and touch the **Setup** button.

Touch the **Vent Setup** button in the bottom-left corner of the touch screen display.

Touch a parameter in the lower margin of the touch screen.

PB 980

MEHIO

- ✓ Swipe to the right to access Setup, Respiratory Mechanics, Stand-by and Screen Capture menus.
- ✓ Touch the Setup button to view the Vent, Apnea, Alarms and More Settings tabs. This is another way to access mode and breath type changes.

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- ✓ Stand-by state can be used to pause ventilation during a planned disconnect of the breathing circuit.
- ✓ Stop spray from circuit during disconnect.
- ✓ Ventilation resumes automatically when the circuit is reconnected to the patient.

Setup:

1. Touch the **Menu** tab on the left side of the touch screen.
2. Touch the **Stand-by** button. A stand-by state pending dialog appears instructing the clinician to disconnect the patient circuit. An on-screen timer allows 30 seconds for disconnect.
3. After you disconnect, you must reconfirm your intent by touching the **Confirm** button within 30 seconds.
4. Flow sensors are monitored to detect patient reconnection.
5. To exit stand-by – reconnect the patient to the ventilator.

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- ✓ Patient-related alarms are temporarily suppressed.
- ✓ The ventilator displays an indicator that it is in stand-by state and a timer indicating the elapsed time the ventilator has been in stand-by state.
- ✓ The exhalation valve is open.
- ✓ Base flow is set to 10 L/min.
- ✓ 100% O₂ is delivered for pediatric or adult application.
- ✓ 40% O₂ is delivered for neonatal application.
- ✓ Entry into and exit from stand-by state is recorded in the general event log.

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
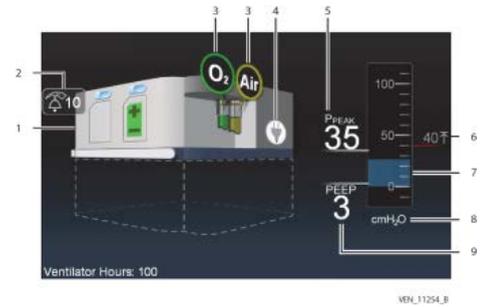
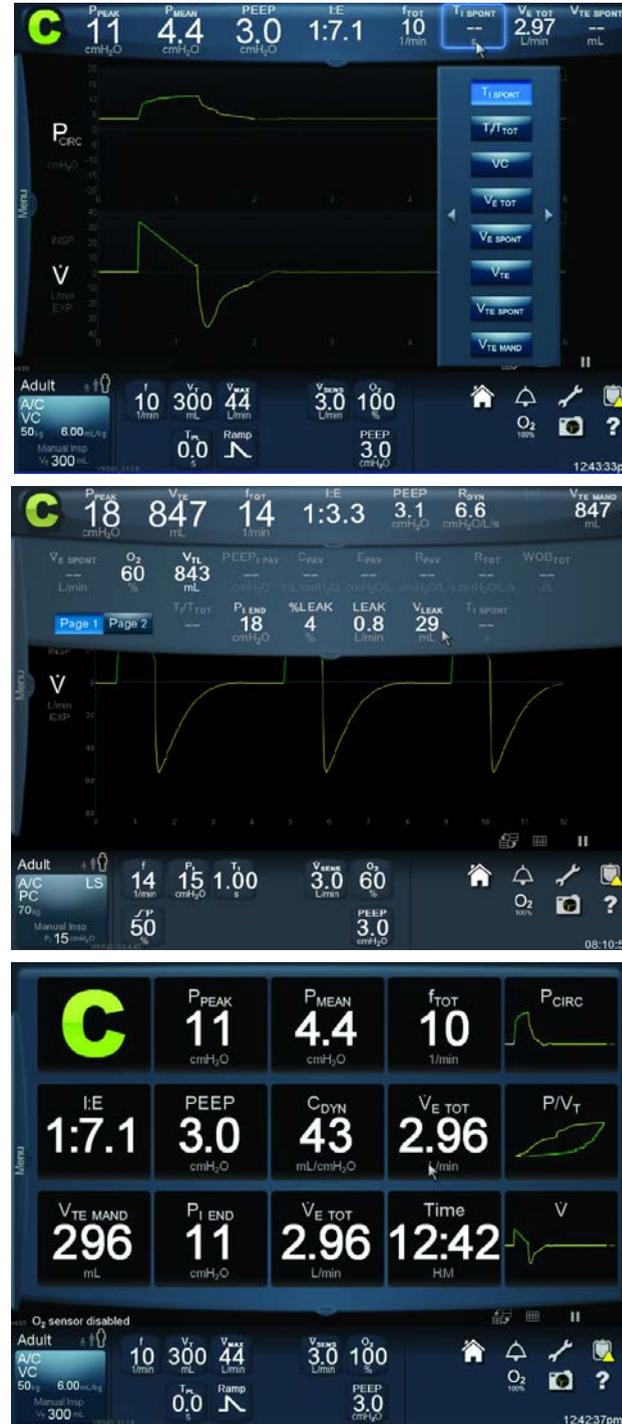
Status display

PB 980

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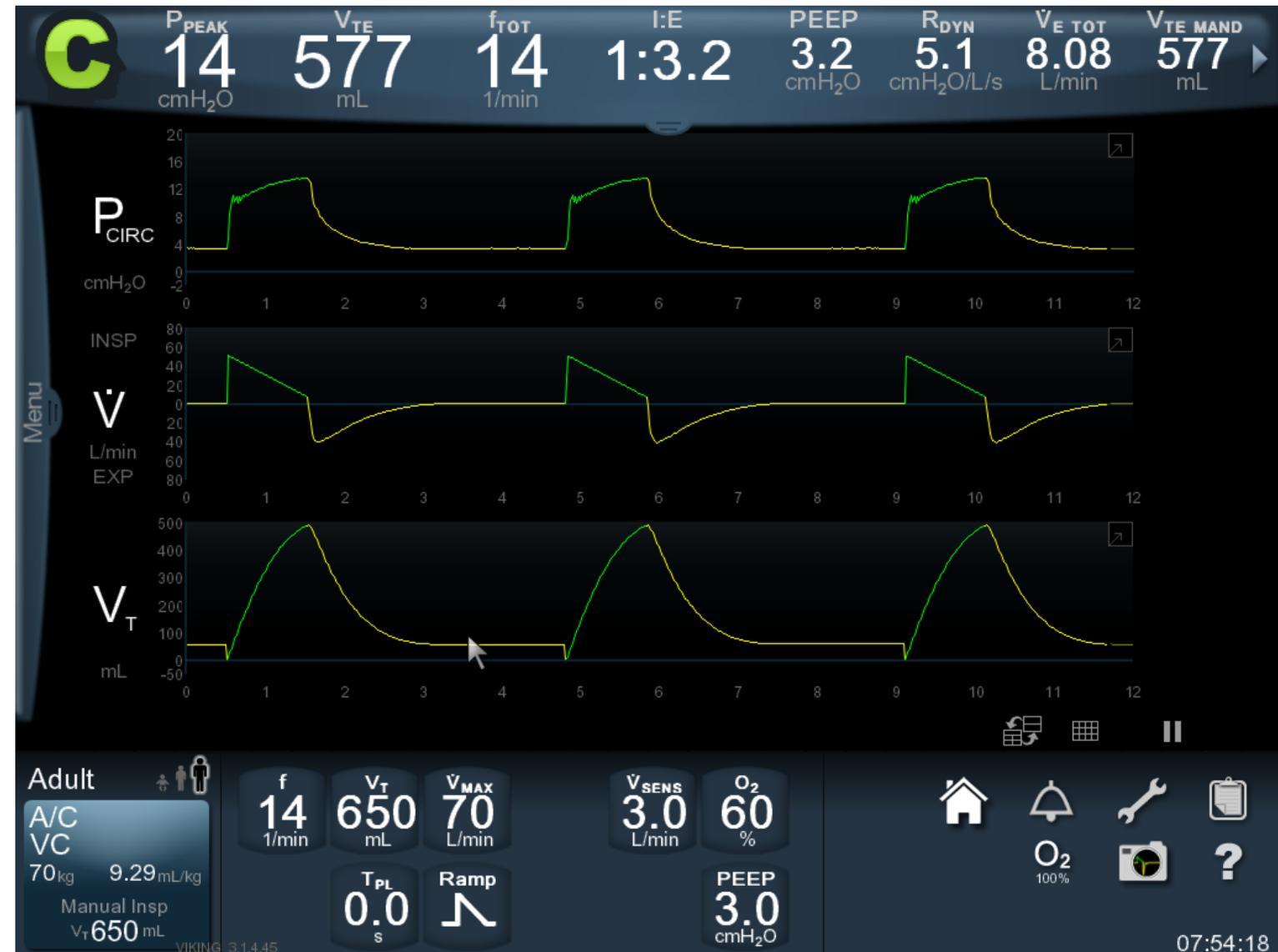- ✓ The Puritan Bennett™ 980 ventilator displays color-coded waveforms for flow, pressure and volume versus time.
- ✓ It also displays color-coded loops for volume versus pressure and flow versus volume.
- ✓ The inspiratory portion of a mandatory breath is green, the inspiratory portion of a spontaneous breath is orange and exhalation is always yellow.

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- ✓ Two ways to change the scale of a graphic:
- ✓ Touch and drag the scale.
- ✓ Touch to select the scale and then use the adjustment knob to change the scale.

- ✓ Graphics can also be enlarged to full screen by double-tapping the graph, swiping upward, or tapping the arrow in the upper-right corner.
- ✓ To return to the previous size, tap the arrow in the upper-right corner of the graph again, swipe downward, or double tap the graph.

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
- ✓ Graphics can be paused and historical data (up to 60 seconds) can be reviewed.
- ✓ Touch the Pause icon in the lower-right corner.
- ✓ Drag or use the adjustment knob to move the cursor and identify measurements along the waveform(s) or loop.
- ✓ Numerical data continues to update while waveform/loop plotting is paused.
- ✓ Touch the Pause icon again to unpaused the graphics.

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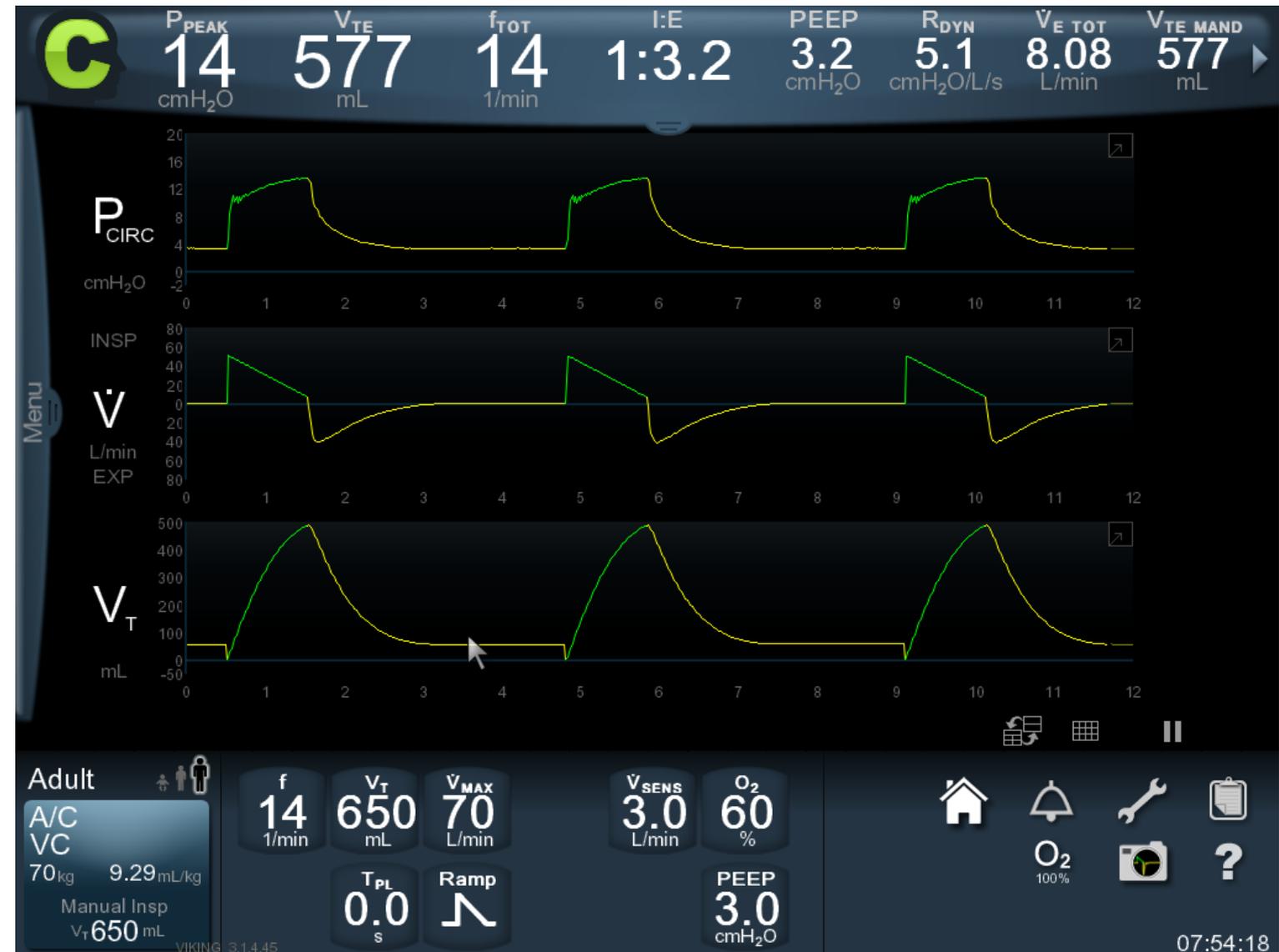
- ✓ Touch the camera icon to store an image of the screen.
- ✓ May be used with or without Pause.
- ✓ Data continues to update during the pause – graph and numerical data on a stored image will not likely be aligned.
- ✓ Open the Menu tab (left margin of screen) and touch Screen Capture to access stored screen images and download through the USB port.

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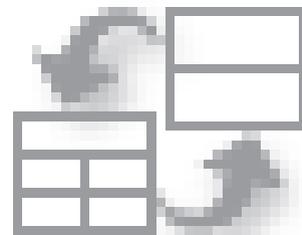- ✓ Use the waveform layout icon to access alternate screen configurations.
- ✓ You can choose to display up to three waveforms and two loops simultaneously in the waveform area.

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- ✓ The patient data log records data every minute for a total of up to 4320 patient data entries.
- ✓ 3 tabs are contained in the patient data log: vital patient data, additional patient data-1, additional patient data-2.
- ✓ During ventilator operation, the log records:
 - Date and time of entry
 - Data label—which measurement it is recording
 - Patient data value
- ✓ Access via the clipboard icon.
- ✓ The log is cleared when the ventilator is set up for a new patient.

The screenshot shows the 'Patient Data' log on the PB 980 ventilator. The log is displayed as a table with the following columns: Time/Date, P_{PEAK} (cmH₂O), V_{TE} (mL), f_{TOT} (1/min), I:E, PEEP (cmH₂O), and P_{MEAN} (cmH₂O). The current entry is highlighted in yellow.

Time/Date	P _{PEAK} cmH ₂ O	V _{TE} mL	f _{TOT} 1/min	I:E	PEEP cmH ₂ O	P _{MEAN} cmH ₂ O
07:22:37pm 16-Dec-2013	22	316	32	6:1	6	19
07:21:37pm 16-Dec-2013	22	316	10	5:1	5	18
07:20:37pm 16-Dec-2013	22	306	25	5:1	5	18
07:19:37pm 16-Dec-2013	22	643	27	1:2	21	19
07:18:37pm 16-Dec-2013	22	344	10	1:5	5	8
07:17:37pm 16-Dec-2013	22	346	17	1:5	5	10
07:16:37pm 16-Dec-2013	22	343	16	1:5	5	9

The interface also shows a sidebar with menu items: Logs, Alarms, Settings, Patient Data (selected), Diagnostics, EST/SST Status, General Event, and Service. At the bottom, there are buttons for 'Vital Patient Data', 'Additional Patient Data - 1', 'Additional Patient Data - 2', 'Export Logs', and 'Close'. The patient type is set to 'Adult' and the time is 04:17:37pm.

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PB 980

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There are three ways to access the Alarms screen:

Touch the hotlink from an alarm violation message.

Swipe the **Menu** tab to open the Settings menu, then touch the **Alarms** settings tab.

Touch the alarm icon (constant access icon).

PB 980

OMNI-DIRECTIONAL LED


Normal mode	✓ Steadily lit green
Alarm condition	✓ LED flashes—color corresponds to alarm priority
Concurrent alarms	✓ LED displays highest-priority color
Alarm de-escalates	✓ Latched (sides) display highest priority ✓ Center displays current alarm priority

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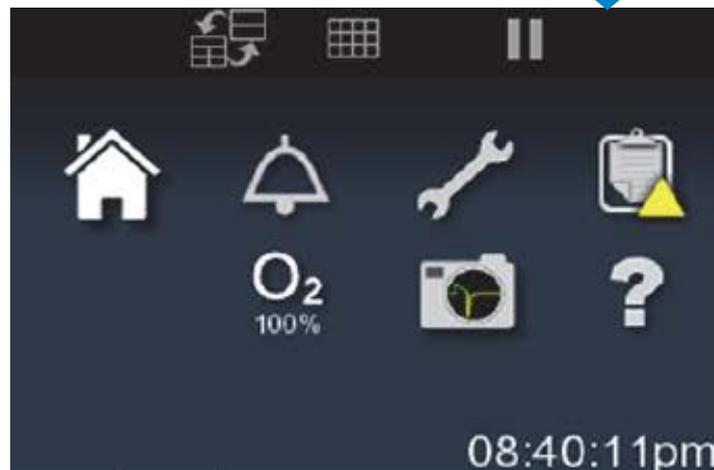Constant access icons	Function
Home icon (house)	Dismisses all open dialogs on the touch screen. Resumes showing ventilator waveforms screen.
Configure (wrench icon)	Opens the configure display to access SST. (history), Options, Comm Setup, and Date/Time change tabs.
Logs (clipboard icon)	Opens the log screen containing tabs for alarms, settings, patient data, diagnostics, EST/ SST status, general event and service logs.
Elevate O ₂ (O ₂ icon)	Increases oxygen concentration to the institutional default O ₂ configuration, if institutional default has been configured, for 2 minutes, or allows the operator to determine the additional percentage of oxygen to increase. Terminate prior to completion of the 2-minute interval by touching Stop .
Screen capture (camera icon)	Captures the image displayed on the touch screen.
Help icon (question mark)	Drag the help (question mark) icon to the item in question and release. A tooltip will appear describing the item's function.

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Waiting for patient connect
Ventilation will begin when connection is detected

Setup Apnea

Ventilation Type: Invasive NIV
Mode: A/C
Mandatory Type: PC VC
Spontaneous Type:
Trigger Type: P-Trig V-Trig

T_A 20 s
Apnea V_E 3.00 L/min

f 10 1/min
 V_T 300 mL
 V_{MAX} 44 L/min
 O_2 100 %
 T_{PL} 0.0 s
Ramp

0.74 1:7.11 5.26 6.00 12 s

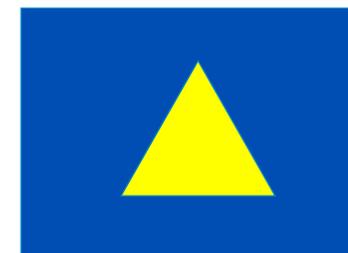
Close Accept

Adult
A/C VC
50 kg 6.00 mL/kg
Manual Insp V_T 300 mL

f 10 1/min
 V_T 300 mL
 V_{MAX} 44 L/min
 V_{SENS} 3.0 L/min
 O_2 100 %
 T_{PL} 0.0 s
Ramp
PEEP 3.0 cmH₂O

08:43:48pm

VEN_11351_D

INDICATORS

PATIENT CIRCUIT INDICATOR (ADULT, PEDIATRIC OR NEONATAL)

- ✓ This indicator appears right above the Vent Setup button.
- ✓ It indicates the size of the patient circuit cleared during the SST.

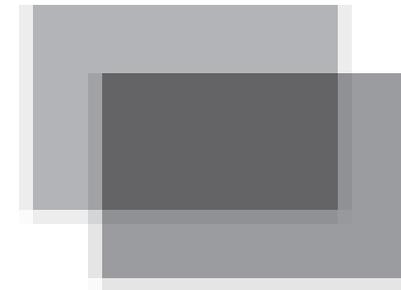
VEN_11351_D

INDICATORS

SCREEN OPACITY ICON


- ✓ Appears on the vent setup screen and on all of the respiratory mechanics maneuvers screens.
- ✓ Select and then use the adjustment knob to adjust the opacity of the displayed information between 50% and 100%.
- ✓ The lower the percentage, the more the waveform display will be visible through the other screen information.

INDICATORS

PUSHPIN ICON

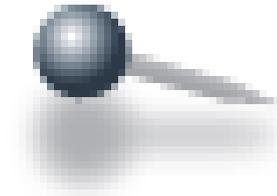✓ Appears in the corner of the settings screen.

✓ Touch the Pushpin Icon to keep a settings window open.

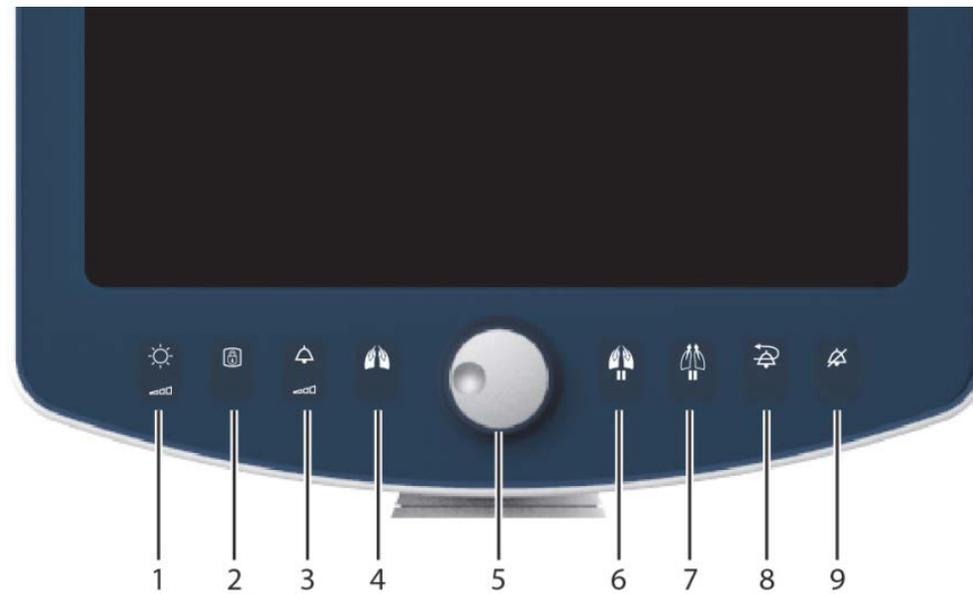
✓ To the reverse the action, unpin or touch the home icon.

USER INTERFACE

BEZEL CONTROL KEYS (OFF SCREEN)


Easy to Access

-  1. Display Brightness
-  2. Display Lock
-  3. Alarm Volume
-  4. Manual Inspiration
- 5. Adjustment Knob
-  6. Inspiratory Pause
-  7. Expiratory Pause
-  8. Alarm Reset
-  9. Alarm Silence

840 VS 980

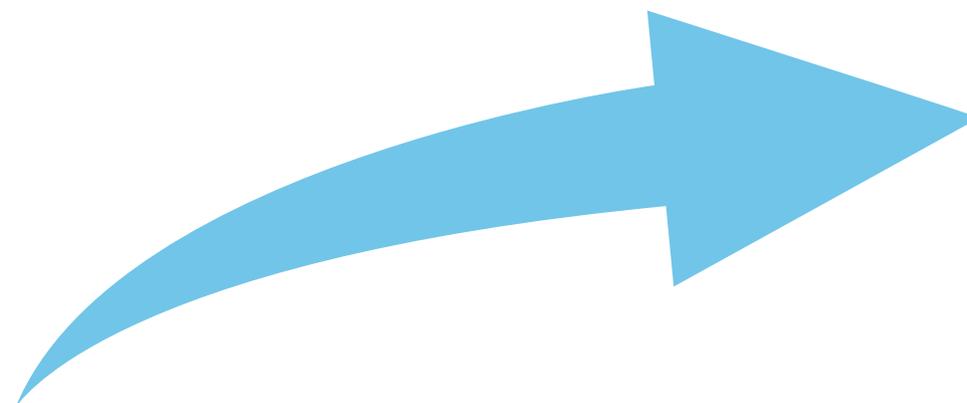RESPIRATORY AND MONITORING SOLUTION

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